

FIRST TO **Know**

SYPHILIS
TEST

UDI CODE, LOT # AND
EXPIRATION DATE HERE

SYPHILIS

Home use fingerstick blood test
for the detection of antibodies
to the bacteria that
cause syphilis



In Home Test
↓
In Home Result
↓
In Minutes

FIRST
TO
Know

1 Test

FIRST
TO
Know

SYPHILIS
TEST


Read the detailed
Instructions For Use
inside before testing.

Go to www.firsttoknow.com
to view comprehensive
Instructions for Use.



TEST OVERVIEW

Prick the fingertip with the
lancet provided. Gently
touch the drop of blood with
the tip of the test until
blood fills the Fill Zone.

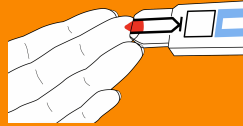

Read the result after 15 minutes
and before 30 minutes.

WHO IS THIS TEST FOR?

This test is for sexually
active people who believe
they have been exposed
to syphilis.

LIMITATIONS

See your healthcare provider
if you have any symptoms
or are pregnant.

Not for people who have
previously had syphilis.

Results will likely be positive
for individuals previously
diagnosed with syphilis,
even if they were
successfully treated.

Anyone with recent sexual
contact with a person known
to have a sexually transmitted
infection should visit a
healthcare provider for
treatment and evaluation
as soon as possible:

www.cdc.gov/std/treatment-guidelines/syphilis.htm

SYPHILIS

Home use fingerstick
blood test for the
detection of antibodies
to the bacteria that
cause syphilis

1 Test



In Home Test >
In Home Result >
In Minutes

FIRST
TO
Know

Store at 59° F - 86° F (15° C - 30° C). Do not freeze.

©2023 NOWDiagnostics

IVD

REF

8057-1

Rev. 1.0

For in vitro diagnostic use.
For over the counter (OTC) use.
Not for internal use.
Single use only.
Do not use if packaging is damaged.
Keep out of the reach of children.

Questions?

Monday-Friday 8AM-5PM CT
Toll Free: 1-844-207-3370
www.firsttoknow.com

CONTENTS:

- One (1) sealed pouch containing one (1) First To Know[®] Syphilis test and desiccant
 - Two (2) single use lancets
 - One (1) alcohol swab
 - One (1) bandage
 - One (1) Instructions For Use in English & Spanish
 - One (1) Educational Guide in English & Spanish
- Other materials needed, but not provided:
• Match or timer

Instrucciones en Inglés y Español

NOWDiagnostics, Inc.
1200 Stewart Place
Springdale, AR 72764 (USA)
Phone: +01 479-966-4530



Patent # 7883899